## MEMINI™ ITE & ITC

with GENIUS™ 3.0 Technology User Manual

# Content

## **Welcome** **4**

---

## **Your hearing aids** **5**

---

Hearing aid type	5
Getting to know your hearing aids	6
Components and names	6
Controls	7
Hearing programs	9
Features	9

## **Batteries** **10**

---

Battery size and handling tips	10
Replacing batteries	11

## **Daily use** **12**

---

Turning on and off	12
Inserting and removing your hearing aids	13
Adjusting the volume	15
Changing the hearing program	15
Further adjustments (optional)	15

## **Special listening situations** **16**

---

On the phone	16
Audio streaming with iPhone	17
Airplane mode	17

## **Maintenance and care** **18**

---

Hearing aids	18
Professional maintenance	19

## **Further information** **20**

---

Safety information	20
Accessories	20
Symbols used in this document	20
Troubleshooting	21
Country-specific information	22
Service and warranty	23

## Welcome

Thank you for choosing Miracle-Ear® hearing aids to accompany you through everyday life. Like anything new, it may take you a little time to become familiar with them.

This guide, along with support from your Hearing Care Specialist, will help you understand the advantages and greater quality of life your hearing aids offer.

To receive the most benefit from your hearing aids, it is recommended that you wear them every day, all day. This will help you get adjusted to them.

### CAUTION

It is important to read this user manual and the safety manual thoroughly and completely. Follow the safety information to avoid damage or injury.

## Your hearing aids

This user manual describes optional features that your hearing aids may or may not have. Ask your Hearing Care Specialist to indicate the features that are valid for your hearing aids.

### Hearing aid type

Your hearing aids are customized models that have been individually manufactured based on the anatomy of your ears. They are worn directly in the ear, either in the bowl of the ear or deep inside the ear canal. They are not intended for children under the age of 3 years or persons with a developmental age of under 3 years.

The wireless functionality enables advanced audiological features and synchronization between your two hearing aids.

Your hearing aids feature *Bluetooth*\*\* low energy technology allowing for easy data exchange with your smartphone, and for seamless audio streaming with your iPhone.\*\*

\*The Bluetooth word mark and logos are owned by the Bluetooth SIG, Inc., and any use of such marks by the legal manufacturer of this product is under licenses. Other trademarks and trade names are those of their respective owners.

\*\*iPhone is a trademark of Apple Inc., registered in the U.S. and other countries.

## Getting to know your hearing aids

It is recommended to familiarize yourself with your new hearing aids. With the hearing aid in your hand try using the controls and note their location on the hearing aid. This will make it easier for you to feel and press the controls while wearing the hearing aids.


**i** If you have problems pressing the controls of your hearing aids while wearing them, you can use a smartphone app to control your hearing aid.

## Components and names


This user manual describes several types of hearing aids. Use the following pictures to identify the type of hearing aid that you use.

Note that the push button and vent are individually positioned on your hearing aid to best suit your ear. The appearance of your hearing aids may therefore differ from that of those pictured.

### ITC (In The Canal)

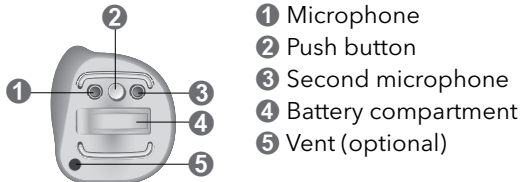### ITE (In The Ear)
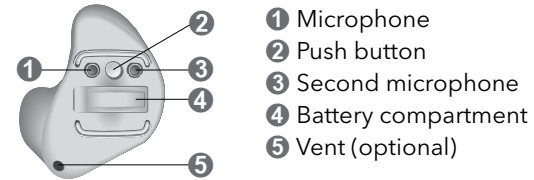
## Controls

With the push button you can, for example, switch hearing programs. Your Hearing Care Specialist has programmed your desired functions to the push button.

**i** You can use a smartphone app for remote control.

## Push button function

	L	R
Press briefly:		
Program up	<input type="radio"/>	<input type="radio"/>
Volume up	<input type="radio"/>	<input type="radio"/>
Volume down	<input type="radio"/>	<input type="radio"/>
Tinnitus control signal level up	<input type="radio"/>	<input type="radio"/>
Tinnitus control signal level down	<input type="radio"/>	<input type="radio"/>
TV stream on/off	<input type="radio"/>	<input type="radio"/>
Press for about 2 seconds:		
Program up	<input type="radio"/>	<input type="radio"/>
Volume up	<input type="radio"/>	<input type="radio"/>
Volume down	<input type="radio"/>	<input type="radio"/>
Tinnitus control signal level up	<input type="radio"/>	<input type="radio"/>
Tinnitus control signal level down	<input type="radio"/>	<input type="radio"/>
TV stream on/off	<input type="radio"/>	<input type="radio"/>
Activate/deactivate airplane mode	<input type="radio"/>	<input type="radio"/>
Press for more than 3 seconds:		
Standby/turn on	<input type="radio"/>	<input type="radio"/>

L = Left, R = Right

## Hearing programs

1
2
3
4
5
6

Read more in section "Changing the hearing program."

## Features

- Power-on delay** enables whistle-free insertion of hearing aids.  
Read more in section "Turning on and off."
- The **tinnitus control** feature generates a sound to distract you from your tinnitus.

## Batteries

When the battery is low the sound becomes weaker or you will hear an alert signal. The battery type will determine how long you have to replace the battery.

### Battery size and handling tips

Ask your Hearing Care Specialist for recommended batteries.

---

Battery size:	312
---------------	-----


---

- Always use the correct battery size for your hearing aids.
- Remove the batteries if you intend not to use the hearing aids for several days.
- Always carry spare batteries.
- Remove empty batteries immediately and follow your local recycling rules for battery disposal.

## Replacing batteries


Removing the battery:

- ▶ Open the battery compartment.
- ▶ Remove the battery by hand.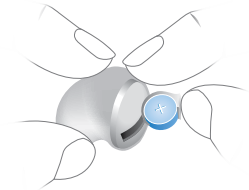

Inserting the battery:

- ▶ If the battery has a protective film, remove it only when you are ready to use the battery. Discard the protective film once it has been removed.
- ▶ Insert the battery with the "+" symbol facing upwards.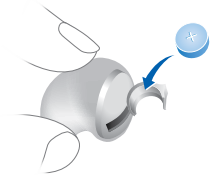

- ▶ Carefully close the battery door. If you feel resistance, the battery is not inserted correctly.

Do not attempt to close the battery door by force. It can be damaged.

## Daily use

### Turning on and off

You have the following options to turn your hearing aids on or off.

Via battery door:

- ▶ Turning on: Close the battery door.  
The default volume and hearing program are set.
- ▶ Turning off: Open the battery door fully.

Via push button:

- ▶ Turning on or off: Press and hold the push button for several seconds. Refer to section "Controls" for the settings of your controls.

After turning on, the previously used volume and hearing program remain set.

When wearing the hearing aids, an alert tone can indicate when a hearing aid is being turned on or off.

When the power-on delay is activated, the hearing aids turn on after a delay of several seconds. During this time, you can insert the hearing aids into your ears without experiencing unpleasant feedback whistling.

The "power-on delay" can be activated by your Hearing Care Specialist.

## Inserting and removing your hearing aids


Your hearing aids have been fine-tuned for your right and left ear. The labeling on the hearing aids indicates the side:

- "R" = right ear
- "L" = left ear

Inserting a hearing aid:

- ▶ Carefully push the hearing aid into the ear canal.
- ▶ Twist it slightly until it sits well.

Open and close your mouth to avoid accumulation of air in the ear canal.

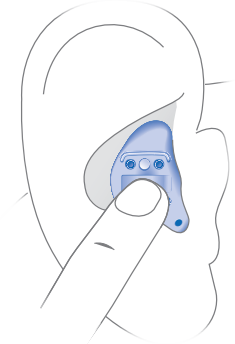
Removing a hearing aid:

### **NOTICE**

Never pull on the battery door to remove the hearing aid. This could damage your hearing aid.

- ▶ Push lightly on the back of your ear to loosen the hearing aid.
- ▶ Grip the hearing aid in the ear canal with two fingers and pull it carefully out.

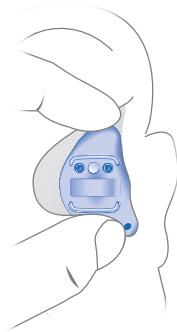Clean and dry your hearing aids after wearing. Read more in section "Maintenance and care."

## **Adjusting the volume**

Your hearing aids automatically adjust the volume to the listening situation.

- ▶ If you prefer manual volume adjustment, press the push button.  
Refer to section "Controls" for the setting of your push button.

An optional signal can indicate the volume change.

## **Changing the hearing program**

Depending on the listening situation, your hearing aids automatically adjust their sound.

Your hearing aids may also have several hearing programs which allow you to change the sound, if needed. An optional signal tone can indicate the program change.

- ▶ To change the hearing program, press the push button briefly.  
Refer to section "Controls" for the setting of your controls. Refer to section "Hearing programs" for a list of your hearing programs.

The default volume is applied.

## **Further adjustments (optional)**

You can also use the hearing aid controls to change other features, for example, the level of the tinnitus control signal.

Refer to section "Controls" for the setting of your controls.

## Special listening situations

### On the phone

When you are on the phone, turn the receiver slightly so that it does not completely cover your ear.

### Telephone program

You may prefer a certain volume when using the phone. Ask your Hearing Care Specialist to configure a telephone program.

- ▶ Switch to the telephone program whenever you are on the phone.

If a telephone program is configured for your hearing aids, it is listed in section "Hearing programs."

### Audio streaming with iPhone

Your hearing aids are Made for iPhone® hearing aids. This means you can get phone calls and listen to music from your iPhone directly into your hearing aids.

For more information about compatible iOS devices, pairing, audio streaming and other useful functions, contact your Hearing Care Specialist.

### Airplane mode

In areas where the use of Bluetooth technology is not allowed (e.g. on some airplanes), you can activate the airplane mode. This turns Bluetooth in your hearing aids off temporarily. The hearing aids still work without Bluetooth, however direct audio streaming will not be possible and other functions will not be available.

- ▶ To activate or deactivate the airplane mode, use the smartphone app or use the controls of your hearing aids.

Refer to section "Controls" for more information.

An alert tone indicates when airplane mode is activated or deactivated.

## Maintenance and care

To prevent damage it is important that you take care of your hearing aids and follow a few basic rules, which will soon become a part of your daily routine.

### Hearing aids


#### Drying and storage

- ▶ Dry your hearing aids overnight.
- ▶ For longer periods of non-use, store your hearing aids in a dry environment with an open battery compartment and the batteries removed.
- ▶ Ask your Hearing Care Specialist for more information.

#### Cleaning

For hygiene reasons and to maintain functionality, clean your hearing aids daily.

- ▶ Clean your hearing aids daily with a soft, dry tissue.
- ▶ Your hearing aids may have a wax protection. This prevents the ingress of earwax. Be careful not to damage the wax protection when cleaning your hearing aids.

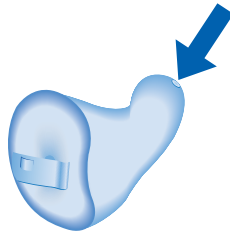- ▶ Never use running water or immerse the devices in water.
- ▶ Never apply pressure while cleaning.
- ▶ Ask your Hearing Care Specialist for recommended cleaning products, special care sets, or more information on how to keep your hearing aids in good condition.

#### Professional maintenance

Your Hearing Care Specialist can perform a thorough professional cleaning and maintenance.

In the event that the wax protection or microphone protection (windscreen) needs to be replaced, this should be carried out by a Specialist.

Ask you Hearing Care Specialist for your individual recommendation of maintenance intervals and for support.

## Further information

### Safety information

For more safety information, refer to the safety manual supplied with the device.

### Accessories

The type of available accessories depends on the hearing aid type. Examples include: a remote control, an audio streaming device or a smartphone app for remote control.

Consult your Hearing Care Specialist for further information.

### Symbols used in this document

---

 Points out a situation that could lead to serious, moderate, or minor injuries.

---

 Indicates possible property damage.

---

 Advice and tips on how to handle your device better.

---

 "Made for iPhone," "Made for iPad," and "Made for iPod" mean that an electronic accessory has been designed to connect specifically to iPhone, iPad, or iPod, respectively, and has been certified by the developer to meet Apple performance standards.

Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards. Please note that the use of this accessory with iPhone, iPad, or iPod may affect wireless performance.

## Troubleshooting

### Problem and possible solutions

---

Sound is weak.

---

- Increase the volume.
  - Gently close the battery door completely.
  - Clean the hearing aid.
  - Replace the empty battery.
  - Ask your Hearing Care Specialist to exchange the wax protection and/or the microphone cover.
- 

Hearing aid emits whistling sounds.

---

- Reinsert the hearing aid until it fits securely.
  - Decrease the volume.
  - Clean the hearing aid.
- 

Sound is distorted.

---

- Decrease the volume.
  - Replace the empty battery.
  - Clean the hearing aid.
  - Ask your Hearing Care Specialist to exchange the wax protection and/or the microphone cover.
-

## Problem and possible solutions

---

Hearing aid emits signal tones.

---

- Replace the empty battery.
- 

Hearing aid does not work.

---

- Turn the hearing aid on.
  - Gently close the battery compartment completely.
  - Replace the empty battery.
  - Make sure the battery is inserted correctly.
  - Power-on delay is active. Wait several seconds and check again.
- 

Consult your Hearing Care Specialist if you encounter further problems.

## Country-specific information

USA and Canada

Certification information for hearing aids with wireless functionality:

GENIUS™ 3.0 ITC, GENIUS™ 3.0 ITE

- HVIN: WP4N1
- Contains FCC ID: SGI-RFM001
- IC: 267AB-WP4N1

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Copyright ©2020 Miracle-Ear, Inc.
Distributed by Miracle-Ear, Inc.
Derechos de autor ©2020 Miracle-Ear, Inc.